


# Visalys® Temp



## Temporary crown and bridge material

### Abridged instructions

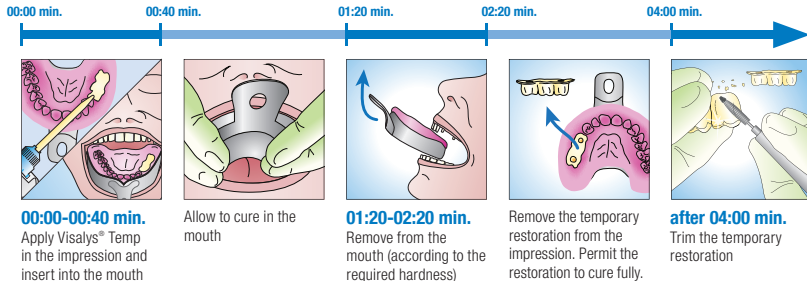

**00:00-00:40 min.**  
Apply Visalys® Temp  
in the impression and  
insert into the mouth

Allow to cure in the  
mouth

**01:20-02:20 min.**  
Remove from the  
mouth (according to the  
required hardness)

Remove the temporary  
restoration from the  
impression. Permit the  
restoration to cure fully.

**after 04:00 min.**  
Trim the temporary  
restoration

\* Neither bisphenol A (BPA) nor bisphenol A derivatives have been used in raw materials, production, filling or packaging.

Distributed in the U.S. by: Kettenbach LP · 400 Oser Ave., Suite 1650 · Hauppauge, NY 11788, USA

# Visalys® Temp



## Temporäres Kronen- und Brückenmaterial Kurzanleitung

